## BIOPSY PUNCH

## EN Instructions for Use

## 1. Device Description

The device is a cutaneous biopsy punch, which consists of a stainless cylindrical hollow tube, "blade", with a cutting edge and plastic handle. The blade diameter varies from 1.0 mm to 8.0 mm.  
The device has a variant as biopsy punch with plunger, which assist to extrude the collected skin tissues inside the blade.

## 2. Instructions for Use

The intended purpose and the clinical benefit of the sterile disposable cutaneous biopsy punch are to perforate and collect skin tissues. The collected tissues are used for examination, treatment, and diagnosis of patients with skin tumors and other skin diseases.  
For use by healthcare professionals only.  
Push and rotate the blade vertically against the subject tissues to perforate and collect them.  
For the device with plunger, push the plunger to extrude the remaining tissues inside the blade.  
Manipulate with attention to the thickness of the skin tissue and other subcutaneous tissue.  
After use, dispose of the device properly as medical waste to prevent infection.

## 3. Contra-indications

Do not use on patients who present sensitisation or allergy.  
As the device contains Co (cobalt), sustained contact may result in adverse effects; use only for the intended purpose.

## 4. Warnings

Reuse of the device presents a risk of infection to the patient.  
Store the device at ambient temperature.  
(Keep packaged devices within temperature limits of -20°C to 60°C during transport and storage.)  
Do not use the device if any damage or other abnormalities are observed.  
Handle with care so as not to injure yourself by touching the edge of the blade.  
If the cutting edge comes in contact with a foreign object, replace it with a new one.  
Any serious incident occurring in relation to the device should be reported to the manufacturer and the competent authority.

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KAI INDUSTRIES CO., LTD.
KAI EUROPE GmbH