DERMAL CURETTE

EN Instructions for use

1. Device Description

The device is a cutaneous biopsy curette, which consists of stainless steel ring, "blade", with a cutting edge and plastic handle. The blade diameter varies from 2.0 mm to 7.0 mm.

2. Instructions for Use

The intended purpose and the clinical benefit of the sterile disposable cutaneous biopsy curette are to curette skin tissue for the examination, diagnosis and treatment of skin disease.

For use by healthcare professionals only.

Push the blade against the subject tissues to curette and collect them.

After use, dispose of the device properly as medical waste to prevent infection.

3. Contra-indications

Do not use on patients who present sensitisation or allergy.

As the device contains Co (cobalt), sustained contact may result in adverse effects; use only for the intended purpose.

4. Warnings

Reuse of the device presents a risk of infection to the patient.

Store the device at ambient temperature.

(Keep packaged devices within temperature limits of -20°C to 60°C during transport and storage.)

Do not use the device if any damage or other abnormalities are observed.

Handle with care so as not to injure yourself by touching the edge of the blade.

If the cutting edge comes in contact with a foreign object, replace it with a new one.

Any serious incident occurring in relation to the device should be reported to the manufacturer and the competent authority.

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KAI INDUSTRIES CO., LTD.
KAI EUROPE GmbH