


# SURGICAL BLADE

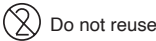

Do not reuse

**【Contraindications and Prohibitions】**

<Applications (Applicable Patients)>

Do not use the product on any patients who are sensitive or allergic to metal.

<Combination with the product>

Do not fix the blade to any handle which is not designed for this product.

(Cf. The section “Interaction”)

<Prohibitions>

Re-use prohibited


Re-sterilize prohibited

Do not use the product when the aseptic condition of the products is considered to be lost due to damage/breakage of the package.

**【Shape and Structure】**

This product is a disposable surgical blade which consists of blade only.

**1. Main Blade Shapes**

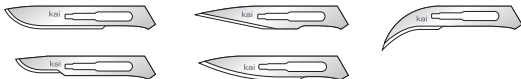

**2. Material**

Blade: Stainless Steel (Chromium contained)

Carbon Steel (Chromium contained)

**【Purpose】**

This product is a blade for surgery. This product is single use.

**【Handling and Use】**

**1. Handling and Use**

Attach the blade to the corresponding blade handle and use the scalpel, following the established surgical method.

**2. Medical Devices which can be used with the combination of the blade**

This product can be used with the combination of “Kai Surgical Knife Handle”.

**【Precautions for Use】**

**1. Important Precautions**

- (1) Do not use the product if the package is damaged/broken (The aseptic condition cannot be assured even though the product is within their expiry date).
- (2) Be careful of the blade when removing the product from the aluminum pouch. If the blade gets contact with anything else, do not use the product and replace it with a new one.
- (3) Do not use the product if any abnormality such as damage/breakage is recognized.
- (4) Handle the products with care in order to avoid being injured by the blades (The blade is so sharp that it would injure).
- (5) After the usage of the products, be aware of the risk of infections and dispose them properly as medical waste with the perspective of preventing infection.

**2. Interaction**

Prohibitions of the combined use (Do not use together)

Medical Device Name	Possible Cause	Reason
Handles which are not designated by the manufacturer.	The risk increases due to the defect of the product.	The safety of the combined usage is not confirmed.

**【Storage and Expiration Date】**

**1. Storage**

- (1) Avoid exposing the product to high temperature, high humidity, direct sunlight and water. Store the product at the ambient temperature.
- (2) Store the product in a clean and dry place without loading anything on the products.
- (3) Do not store the product together with chemical products or in an area where gas occurs.
- (4) Avoid damaging or making pinholes to package materials.

**2. Expiration Date**

Check the expiry date which is indicated on the individual and external package.

**【Manufacturer】**

KAI INDUSTRIES CO., LTD.  
1110 Oyana, Seki-shi, Gifu 501-3992, Japan