
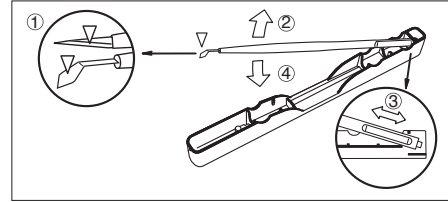
MICROSURGERY KNIFE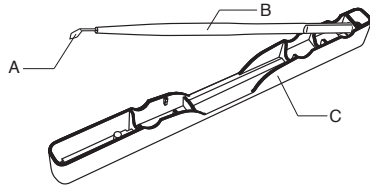

EN Instructions for use

1. Device Description

The device is an ophthalmic microknife, which consists of a stainless steel, mainly sagittal or round-shaped blade (A) with a cutting edge and a plastic handle (B), which is placed in a case (C). There are straight and angled types depending on the attachment of the blade to the handle. There are about 80 blade variations in total.

2. Instructions for Use

The intended purpose and the clinical benefit of the sterile disposable ophthalmic microknife are to incise or detach eye tissue for the treatment of eye diseases.

For use by healthcare professionals only.

Incise or detach the eye tissue.

Lift the handle (2) to remove. To store, insert the back of the handle into the case with the blade tip facing down (1 and 3) and put the handle back in the case (4).

Mind the position of the blade tip when removing it from the case and storing it. After use, dispose of the device properly as medical waste to prevent infection.

3. Contra-indications

Do not use on patients who present sensitisation or allergy.

4. Warnings

Reuse of the device presents a risk of infection to the patient.

Store the device at ambient temperature.

(Keep packaged devices within temperature limits of -20°C to 60°C during transport and storage.)

Do not use the device if any damage or other abnormalities are observed.

Handle with care so as not to injure yourself by touching the edge of the blade.

If the cutting edge comes in contact with a foreign object, replace it with a new one.

Any serious incident occurring in relation to the device should be reported to the manufacturer and the competent authority.

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KAI INDUSTRIES CO., LTD.
KAI EUROPE GmbH