
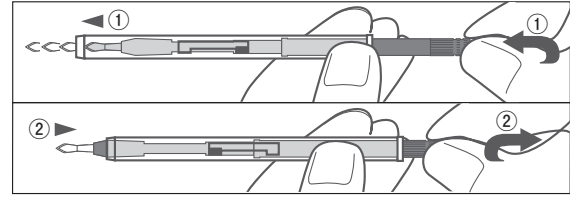
SAFETY KNIFE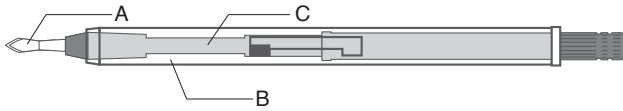

EN

Instructions for use

1. Device Description

The device is an ophthalmic microknife, which consists of a stainless steel, mainly sagittal or round-shaped blade (A) with a cutting edge, a plastic slider (C) to which the blade is attached and a plastic handle (B) which contains it. There are straight and angled types depending on the attachment of the blade to the handle. There are about 30 blade variations in total.

2. Instructions for Use

The intended purpose and the clinical benefit of the sterile disposable ophthalmic microknife are to incise or detach eye tissue for the treatment of eye diseases.

For use by healthcare professionals only.

Incise or detach the eye tissue.

Rotate the slider clockwise to extend and retract the blade (① and ②).

Mind the blade tip while extending the blade and make sure it is secured.

After use, store the blade in the handle and make sure it is secured.

The handle has a groove that allows the slider to move. Do not place your finger on this part when moving the slider to extend or retract the blade.

After use, dispose of the device properly as medical waste to prevent infection.

3. Contra-indications

Do not use on patients who present sensitisation or allergy.

4. Warnings

Reuse of the device presents a risk of infection to the patient.

Store the device at ambient temperature.

(Keep packaged devices within temperature limits of -20°C to 60°C during transport and storage.)

Do not use the device if any damage or other abnormalities are observed.

Handle with care so as not to injure yourself by touching the edge of the blade.

If the cutting edge comes in contact with a foreign object, replace it with a new one.

Any serious incident occurring in relation to the device should be reported to the manufacturer and the competent authority.

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KAI INDUSTRIES CO., LTD.
KAI EUROPE GmbH